VERMEIREN

Quava

INSTRUCTION MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
MANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Rollator - ISO 9999:12 06 06

This page is intentionally left blank

INSTRUCTION MANUAL

Rollator QUAVA

Contents

1	Preface.....	1
2	Your product.....	1
3	Before use.....	2
4	Using your rollator	3
5	Maintenance.....	5
6	Disposal	6
7	Technical details.....	6

8. Wheel (4x)
9. Frame
10. Brake (2x)
11. Walking stick holder
12. Frame cross
13. Folding mechanism
14. Velcro straps
15. Location of the identification plate

1 Preface

First of all we wish to thank you for the trust you placed in us by selecting a product of **Vermeiren**.

Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints would keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.



Visually impaired people can download the electronic version of this manual and by use of a Text-to-speech software application they can have it read out.

2 Your product

2.1 Components (Figure 1)

1. Brake lever
2. Handgrip
3. Button for height adjustment
4. Strap on seat
5. Backrest belt
6. Seat
7. Shopping bag

Figure 1: Rollator components

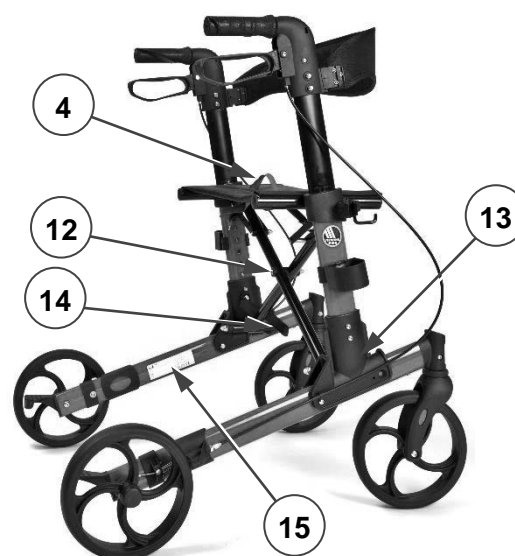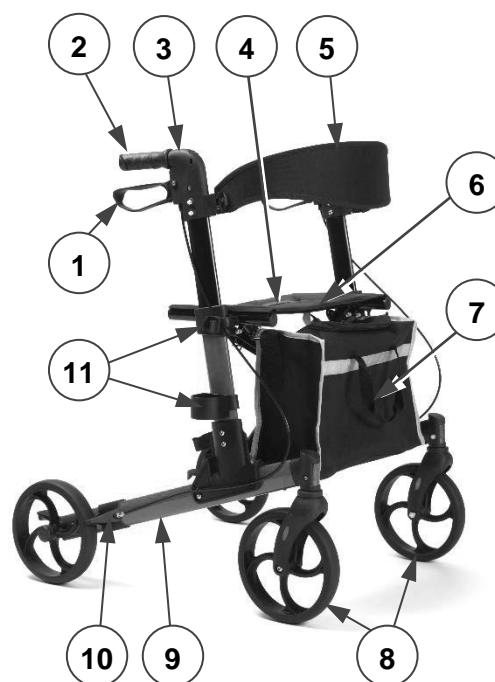2.2 Declaration of conformity

This product is certified for CE and is classified as a Medical product, Class I.

The CE mark indicates the conformity with standards for protection of health, safety and the environment for products traded within the European economic area.

A copy of the Declaration of conformity is available via our website:

<http://www.vermeiren.com/>.

2.3 Lifespan

The expected lifespan of this product is five years, when used according to the instructions in this manual and the service manual. The effective lifespan depends on the use intensity and frequency.

2.4 Warranty

The warranty of this product is based on normal use and maintenance as described in this instruction and relevant documents. In case of damage to your product caused by misuse or lack of maintenance, this warranty will lapse.

3 Before use

3.1 Upon delivery


Unpack your product and verify if the delivery is complete. Following items should be included:

- Rollator frame with seat, 4 wheels, 2 brake systems and 2 adjustable handgrips
- Shopping basket
- Walking stick-holder (1 set)
- Backrest belt
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter.
- Make a report of the damage.
- Contact your supplier immediately.

3.2 Intended use

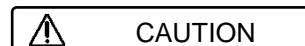Risk of falling

The rollator is meant as support of the leg function during walking. Do not use the rollator if you have other physical limitations that may cause unsafe use of your rollator, like a weak arm, balance disorders,...

The rollator is designed and produced solely to be used as walking aid for one person with a maximum weight of 136 kg.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide better stability, support and safety during walking.

It is not intended for transport of goods and/or more people than the user.

The rollator can be used indoor and outdoor.

It should only be used on surfaces where all four wheels are touching the ground. Do not use the rollator on rocky, bumpy or sandy terrain, or places with a steep inclination. The rollator may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your rollator.

Please check all technical details and limits of your rollator in §7.

3.3 Safety instructions

Please read and follow the

instructions in this manual. Otherwise you may get injured or your rollator may get damaged.

- Check that no stones or other objects block the wheels of the rollator.
- Make sure that both brake levers are in parking mode before you use the rollator as aid to stand up. When you are standing comfortable and stable, you may unlock the parking brakes.
- While proceeding, keep both hands on the handgrips.
- Be aware that some parts of your rollator may get very hot or cold due to

ambient temperature, solar radiation, heating devices, etc.. Be careful when touching.

- Before every usage, check that the height adjustment are firmly secured, see §4.2.
- Verify if the safety against unwanted folding/unfolding is functioning properly, see also §4.3.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

3.4 Symbols on the rollator

Maximum weight of user

Type designation


3.5 Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place
 - with a temperature between +5°C and +41°C
 - with a relative humidity of the air between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- Make sure no objects are stored on or against the rollator that can damage it.

4 Using your rollator

4.1 Unfolding / folding

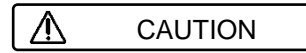CAUTION

Risk of entrapment.

Keep fingers, buckle and clothes away from the folding mechanism and frame cross.

CAUTION

Risk of falling
Check that the

rollator is completely open and that the frame is locked.

Unfolding rollator

1. Place the package with all four wheels on the floor and with the handgrips in front of you, see figure 3.
2. Raise both handgrips to fully upright position and spread the handgrips apart, see figure 2.

Figure 2: Unfolding rollator


- i** Make sure that the handgrips are fully upright before spreading the seat. If not, the frame cannot be opened completely. If this is the case, pull the seat up again to make the frame close and repeat step 2.
3. Push with both hands the sides of the seat down. Keep pushing on the frame tubes till the frame is locked; a click can be heard during locking.
4. Check if the frame is locked, by pushing the handgrips to each other.
5. Adjust the handgrips to the desired height (see details in §4.2).


Folding rollator

1. Remove walking stick and shopping bag.
2. Pull the seat upward by red strap (4) (figure 1).
3. Push handgrips (2) to each other, to close the frame.
4. When the frame is in the smallest position, move the handgrips down (figure 3).
5. Connect top part and lower part of the frame with the Velcro straps (14).

Figure 3: Folding rollator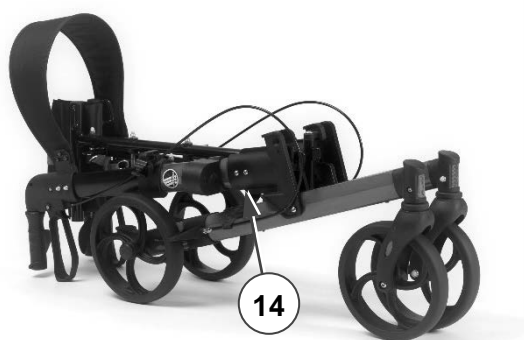


4.2 Adjusting height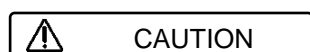

CAUTION

Risk of falling

- Do not exceed the maximum height adjustment of the handgrips, your rollator will not be stable anymore.
- Check the handgrips to be locked before using the rollator.

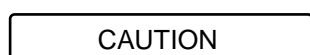CAUTION

Risk of damage to the brakes

Don't pull the handgrips too high to prevent damage to the brake cable.

The handgrips can be adjusted in height by following steps (figure 4):

1. Put your rollator in parking mode, see §4.3.
2. Press and hold adjustment button (3) on the handle.
3. Adjust the height of handgrip by raising or lowering it (6 different positions: in steps). Release button (3) to lock the height adjustment.

i When standing straight, the handgrips should be as high as your wrists. Leave your arms hanging loose.


4. Pull the handgrip up and push it down to check if it is locked.

5. Adjust the other handgrip in the same way.
6. Check that both handgrips are set on the same height mark.

Figure 4: Adjusting the handgrips

4.3 Brake system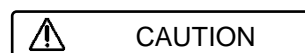


CAUTION

Risk of falling. Check the working of the brakes to ensure your safety. The condition of the braking system is influenced by wear, oil and other contamination of the rear wheels and braking mechanism

The rollator is equipped with brakes, operated by two brake levers (1), see figure 5. The brake levers can be used in two braking modes: running brakes and parking brakes.

Figure 5: Use of brake lever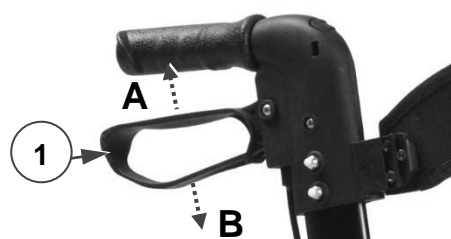

- To use the running brakes, pull both brake levers (1) simultaneously upwards (A in figure 5). To stop braking, release the brake levers.

- To use the parking brakes, push both brake levers (1) downwards in locked position (B in figure 5). Always use both parking brakes simultaneously. To end the parking mode, pull both brake levers simultaneously upwards.

4.4 Backrest belt and stick holder


 CAUTION Risk of falling

Before using the seat:

- Make sure that backrest belt (5) is well secured to the frame.
- Check that all 6 bolts (16) and nuts of belt (5) are well tightened.

Belt and holder easily be mounted by pushing its base over the protruding part of the support on the frame, see example in figure 6. Push it down till it sits tight.

Figure 6: Sliding the backrest belt on/off

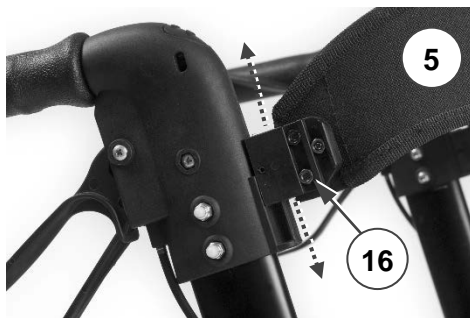To demount, slide the base upwards again till it slides from the support.

4.5 Shopping bag

 CAUTION Risk of falling, tipping over

Do not overload the shopping bag.

The maximum carrying weight of the shopping bag can be found in §7. The shopping bag is meant to be mounted on the hooks below the seat. For this the bag is equipped with two loops, one at each side of the bag.

5 Maintenance

- i** Regular care ensures that your rollator is preserved in a perfectly functional condition.

5.1 Cleaning

Use a soft, damp cloth and lukewarm water with a mild soap for cleaning of the rollator. Avoid soaking.

Do not use abrasive cleaning agents to clean.

5.2 Disinfection

 CAUTION Risk of damage to the rollator

Disinfections may only be undertaken by trained persons. Consult your specialist dealer.

5.3 Inspection / Service / Repairs

The lifespan of the rollator is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the rollator, it should be inspected on following points:

- Completeness.
- Folding mechanism.
- Bolt and nuts regularly; on tightness.
- Check castors (stability, tracking, fixing screws).
In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrip (safe locking of height adjustment system, good support of handgrips).
- Seat, backrest belt and shopping bag (wear or damage).
- Being clean.
- Brakes: check regularly the working of the brakes.

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

i The last page of this manual contains the service registration list to be used by specialist dealer. Additional technical information for the specialist dealer is available via our website:

<http://www.vermeiren.com/>

6 Disposal

Packaging materials can be disposed in the recycling centre in your neighbourhood.

At end of life, you need to dispose your rollator according to the local environmental legislation. The best way to do so, is to disassemble the rollator to facilitate the transport of recyclable parts.

7 Technical details

 CAUTION Risk of unsafe use

Be aware of the correct settings and limits of use like described in this manual

Make	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout Belgium
Type	Rollator 
Model	Quava

<i>Description</i>	<i>Dimensions</i>
Maximum user weight	136 kg
Maximum weight of basket load	5 kg
Dimensions unfolded L x W X (H)	710 x 655 x (830-955) mm
Dimensions folded (handgrips in lowest position) L x W X H	910 x 280 x 340 mm
Distance between handgrips, unfolded	490 mm
Seat height	540 mm
Seat width	450 mm
Seat depth	225 mm
Total mass (with shopping bag and backrest belt)	7,7 kg
Wheels	200 x 35 mm (4x)
Brakes	Running & parking brakes
Max. inclination forwards	16°
Max. inclination backwards	8,5°
Max. inclination sideways	4°
Turning width	895 mm
Storage temperature	+5°C - +41°C
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg / 1,5°.	

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All rights reserved, including translation.
Vermeiren Suisse S.A.