# VERMEIREN

## Eco-Plus

INSTRUCTION MANUAL  
MANUEL D'UTILISATION  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
MANUALE DI ISTRUZIONI  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI  
NÁVOD K OBSLUZE

EN

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*Chodítko - Rollator*

Rollator – ISO 9999: 12 06 06

This page is intentionally left blank

## INSTRUCTION MANUAL

### Rollator Eco-Plus

#### Contents

1	Preface.....	1
2	Your product.....	1
3	Before use.....	2
4	Assembly and disassembly .....	3
5	Using your rollator .....	6
6	Maintenance.....	7
7	Disposal .....	8
8	Technical details.....	8

#### 1 Preface

First of all, we wish to thank you for the trust you placed in us by selecting a product of Vermeiren.

Before using the rollator, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your rollator. Please remember that observance of our hints will keep your rollator in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

Visually impaired people can download the electronic version of this manual and by use of a text-to-speech software application they can have it read out.


#### 2 Your product

##### 2.1 Components (Figure 1)

1. Brake lever
2. Handgrip
3. Brake cable
4. Backrest
5. Velcro strap (walking stick)
6. Seat

7. Fastener pin
8. Front-wheel frame
9. Rear wheels
10. Brake
11. Storage bag
12. Knob for height adjustment
13. Front wheels
14. Tipping lever
15. Cable clamps
16. Silvery push button
17. Walking stick holder
18. Base frame

Figure 1: Rollator components

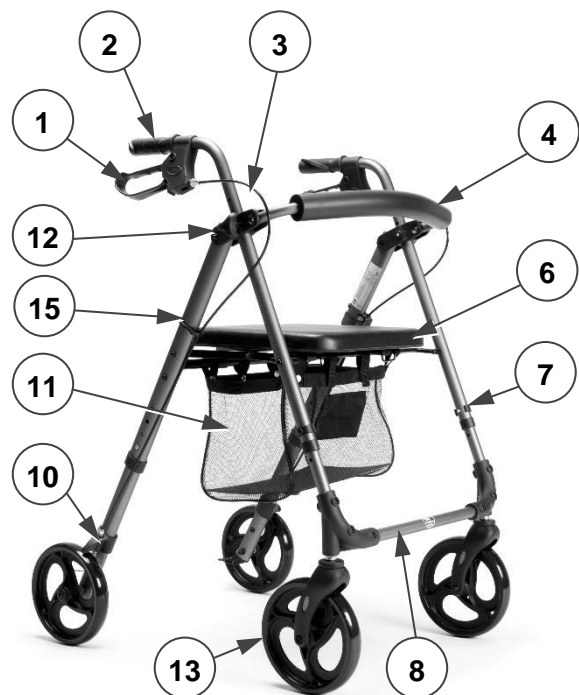## 2.2 Declaration of conformity

This product is certified for CE and is classified as a Medical device, Class I. 

The CE mark indicates the conformity with standards for protection of health, safety and the environment for products traded within the European economic area.

A copy of the Declaration of conformity is available via our website:

<http://www.vermeiren.com/>.

## 2.3 Lifespan

The expected lifespan of this product is reached when used according to the instructions in this manual and the service manual. The effective lifespan depends on the use intensity and frequency.

## 2.4 Warranty

The warranty of this product is based on normal use and maintenance as described in this instruction and relevant documents. In case of damage to your product caused by misuse or lack of maintenance, this warranty will lapse.

## 3 Before use

### 3.1 Upon delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- Base frame with seat and backrest;
- Front-wheel frame with front wheels;
- Two combined parts (for left and right side): wheel support with rear wheel, brake, brake cable, cable clamp and handgrip. The left wheel support comprises a walking-stick holder;
- Two height-adjustment knobs;
- Two fastener pins;
- Storage bag;
- One Velcro strap;
- Manual.

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- Contact the transporter.
- Make a report of the damage.
- Contact your supplier immediately.

### 3.2 Intended use

 CAUTION Risk of falling

**The rollator is meant as support of the leg function during walking. Do not use the rollator if you have other physical limitations that may cause unsafe use of your rollator, like a weak arm, balance disorders, ...**

The rollator is designed and produced solely to be used as walking aid for one person with a maximum weight of 136 kg.

The rollator will compensate the lack of adequate support by the legs by leaning on, or moving with, the rollator with usage of the upper limbs or chest. The rollator will provide better stability, support and safety during walking.

The rollator can be used indoor and outdoor.

It is not intended for transport of goods and/or more people than the user.

It should only be used on surfaces where all four wheels are touching the ground. Do not use the rollator on rocky, bumpy or sandy terrain, or places with a steep inclination. The rollator may become unstable. Look for flat, stable areas, and bevelled kerbs to walk with your rollator.

Please check all technical details and limits of your rollator in §8.

### 3.3 Safety instructions

 CAUTION **Please read and follow the instructions in this manual. Otherwise you may get injured or your rollator may get damaged.**

- Check that no stones or other objects block the wheels of the rollator.
- Make sure that both brake levers are in parking mode before you use the rollator as aid to stand up. When you are standing comfortable and stable, you may unlock the parking brakes.
- While proceeding, keep both hands on the handgrips.
- Be aware that some parts of your rollator may get very hot or cold due to

ambient temperature, solar radiation, heating devices, etc. Be careful when touching.

- Before every usage, check that the height adjustments are firmly secured, see §4.1 and 4.2.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not proceed with your rollator into traffic. Always stay on the pavement.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing / care.

### 3.4 Symbols on the rollator

**i** Identification plate is located beneath the left handgrip at the inner side of the base frame

Maximum weight of user

Type designation

### 3.5 Shipping and storage

The shipping and storage of the rollator shall be according following instructions:

- Store in a dry place
  - with a temperature between +5°C and +41°C
  - with a relative humidity of the air between 30% and 70%.
- Provide sufficient covering or packaging to protect the rollator from rust and foreign bodies (e.g. salt water, sea air, sand, dust).
- Make sure no objects are stored on or against the rollator to prevent damage.

## 4 Assembly and disassembly

CAUTION


Risk of injuries

**Assemble the rollator completely, before adjusting its height.**

### 4.1 Assembly of front-wheel frame

1. Unfold base frame (18).
2. Place the base frame on ground or table, with the oval tubes flat on the surface.
3. Take front-wheel frame (8) and insert both tube ends in the protruding, round tubes of the base frame. Make sure that the wheel support (w) is on the inside of the base frame.


Figure 2: Front wheel frame, inserted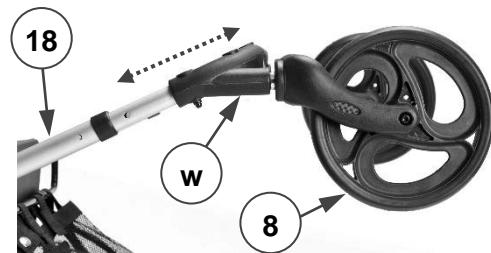

4. Adjust the tube length of the front-wheel frame (8) by sliding the tubes in/out.
5. Check on each side, whether the holes of inner and outer tubes match.

Figure 3: Fastener pin, inserted

6. Insert fastener pin (7) in the hole of the height adjustment. Make sure that the pin is completely inserted; a click is heard when fastener clip clasps the tube.
7. Repeat this at the other frame tube.
8. Check the tube length at left and right side. If the length differs, remove the fastener pin at one side and adjust the length according to step 4, 5 and step 6.

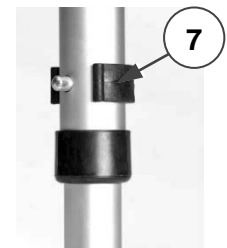## 4.2 Assembly of rear wheels and handgrips

**CAUTION** Risk of damage  
Prevent brake cables of being folded.


*Wheel supports with rear wheels*

1. Turn the base frame till the front-frame tubes (round) are at the bottom, with the front-wheels touching the surface.

*Figure 4: Combined parts*

2. Take one of the combined parts with rear wheel, handgrip and braking cable. 
3. Before assembly, check if rear wheel (9) and brake (10) will be located on the outside of the oval frame tube. If this is not the case, left and right wheel support have been switched. Use this wheel support at the other side of the base frame.

*Figure 5: Silvery push button*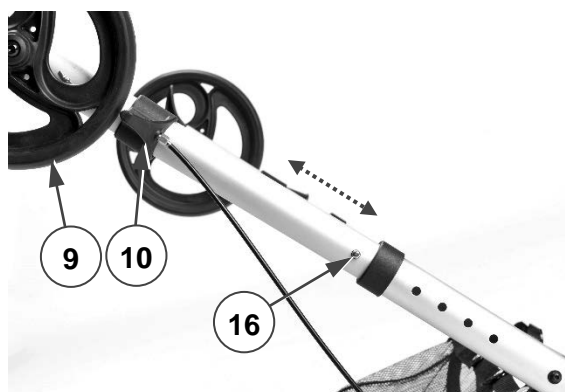

4. Press and hold the small, silvery push button (16), see figure 5.
5. Insert the wheel support in the protruding, oval tube of base frame.
6. Slide the wheel support in/out till push button (16) pops out of the adjustment hole.
7. Press the push button again and slide the wheel support to the desired length.
8. Repeat this for the second wheel support (of combined part). Make sure that left and right tube lengths are the same.
9. Make sure that both tubes are well secured.


*Handgrip with brake*

**CAUTION** Risk of falling  
Check before use if handgrips are locked.

**CAUTION** Risk of damage to the brakes  
Pay attention to the position of the brake cables: they must be on the outside of the frame without interfering the rear wheels.


Don't pull the handgrips too high to prevent damage to the brake cable.

*Figure 6: Handgrips height adjustment*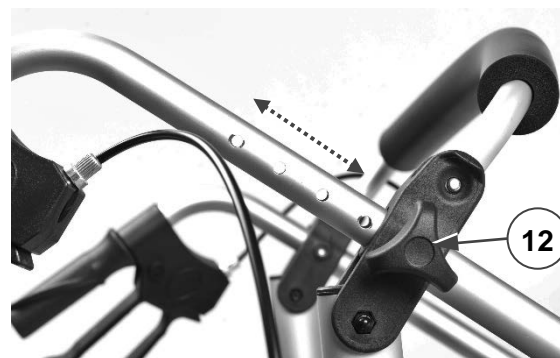

1. Turn the base frame and place it with all four wheels on the ground.
2. Insert the tube with handgrip in the open tube end of the frame, at the connection piece.
3. Place knob (12) in the holes of the tube and tighten firmly by hand.
4. Mount the other handgrip in the same manner.

*Figure 7: Securing brake cables*

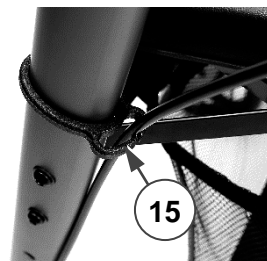Note: the cable clamp is already on the brake cable.

5. Slide cable clamps (15) up while guiding the cable.
6. Keep the brake cable aligned with the rear frame tube and push the clamp round the frame tube, see picture.

### 4.3 Height adjustments

- i** To adjust the height of the rollator:
1. First adjust the seat height.
  2. Next adjust the height of both handgrips.

#### Height of seat

**CAUTION** **Risk of falling**  
**Make sure that both brake levers are in parking position before sitting down, see §5.1.**

1. Turn the base frame and place it with all four wheels on the ground.
2. Turn seat and backrest down.
3. Check whether the seat is horizontal. If not, adjust the assembled wheels (see §4.1 and 4.2) at the same height.
4. Sit down on the seat according to §5.3.
5. Check whether the seat is at the correct sitting height. If not, adjust the assembled wheels (see §4.1 and §4.2) and make sure that they are at the same height.

#### Height of handgrips

1. Loosen knob (12) (see §4.2) and remove from frame.
2. Adjust the handgrip to the desired height. When standing straight, the handgrip should be as high as your wrist. Leave your arms hanging loose.
3. Place knob (12) in the holes of the tube and tighten firmly by hand.
4. Adjust the other handgrip in the same manner. Make sure that both handgrips have the same height.

### 4.4 Mounting of storage bag

**CAUTION** **Risk of falling, tipping over**  
**Do not overload the storage bag.**

The maximum carrying weight of the storage bag can be found in the table of §8.

The storage bag is meant to be mounted below the seat, on the two horizontal tubes where the seat rests on.

1. Pull seat frame (23) a bit up to make room.
2. Mount the storage bag with the six snap fasteners (20) and two Velcro straps

- (21). Make sure that the two protection sleeves (22) stay uncovered.
3. Push seat and seat frame down.

**Figure 8: Storage bag assembly**

### 4.5 Disassembly

**CAUTION** **Risk of damage to the brakes**  
**Pay attention to the position of the brake cables and don't pull the handgrips too high to prevent damage to the brake cable.**

The disassembly of the rollator is conducted in reverse order to the assembly described in previous section:

1. Storage bag: Open snap fasteners and Velcro straps to remove the bag.
2. Velcro strap for walking stick: Remove.
3. Cable clamp (2x): Pull it from the frame tube carefully.
4. Handgrip with brake lever (2x): Remove knob (12) and pull the handgrip out of the base frame.
5. Base frame: Turn and place it with the front frame on a surface.
6. Wheel support with rear wheel (2x): Press and hold silvery push button (16) and pull the wheel support out of the base frame.
7. Base frame: Turn and place it with the rear frame on the surface.
8. Front-wheel frame (8): Remove on each side, fastener pin (7) and pull the tubes out of the base frame.

## 5 Using your rollator

### 5.1 Brake system

 CAUTION **Risk of falling. Check the working of the brakes to ensure your safety. The condition of the braking system is influenced by wear, oil and other contamination of the rear wheels and braking mechanism.**

The rollator is equipped with brakes, operated by two brake levers (1), see next figure. The brake levers can be used in two braking modes: running brakes and parking brakes.


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
 CAUTION **Risk of damage. Do not push the rollator to hard while using the running brakes. This may cause damage to tires or brakes.**

Figure 9: Use of brake lever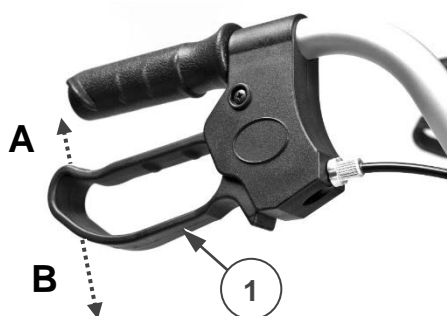

- To use the running brakes, pull both brake levers (1) simultaneously upwards (A in figure 9) in a squeezing movement. To stop braking, release the brake levers.
- To use the parking brakes, push both brake levers (1) downwards in locked position (B in figure 9). Always use both parking brakes simultaneously. To end the parking mode, pull both brake levers simultaneously upwards.

### Unfolding / folding

 CAUTION **Risk of entrapment. Keep fingers, buckle and clothes away from the folding mechanism.**

 CAUTION **Risk of falling. Check that the rollator is completely open and that the frame is locked.**

### Unfolding rollator (Figure 10)

1. Push both brake levers down to parking position, see §5.1.
2. Place the rollator with the rear wheels on the ground while holding the handgrips.
3. Push seat and seat frame (23) down to horizontal position.
4. Turn the backrest down, if it is up.
5. Adjust the rollator to the desired height (see §4.3).

Figure 10: Unfolding rollator

### Folding rollator

1. Stand behind the rollator.
2. Push both brake levers down to parking position, see §5.1.
3. Turn the seat upward.
4. Empty the storage bag.
5. Pull the seat frame (23) upward, to close the frame a bit (in opposite direction to the direction in figure 10).
6. Close the frame completely by pushing front and rear tubes together.
7. If necessary, turn the backrest upwards.

## 5.2 *Sitting down, standing up and walking*

 **CAUTION** **Risk of falling**  
**Make sure that both brake levers are in parking position before sitting down. Keep the brake levers in this position till you start walking again.**

**Keep your feet between the rear wheels.**


1. Push both brake levers down to parking position.
2. Check whether the backrest and seat are turned down.
3. While leaning on the handgrips, turn around and sit down on the seat.
4. To stand up again, lean on the handgrips and turn around.
5. Pull both brake levers up and use the running brakes.
6. Release the brake levers and step forward with the rollator close to your body.

## 5.3 *Negotiating kerbs*

 **CAUTION** **Risk of falling**  
**Take kerbs straight, with both front wheels aligned to the kerb. Keep the rollator nearby.**

1. Walk till your rollator stands in front of the kerb (upwards).
2. Make sure you are standing stable while holding both handgrips.

**Figure 11: Using tipping lever (14)**

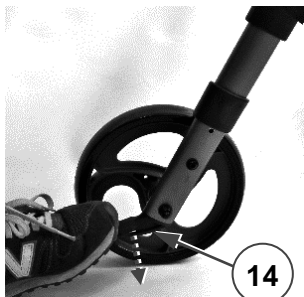3. Use one foot and push on tipping lever (14) to lift the front wheels.
4. Keep the rollator in that position with the handgrips and move slowly forward till the front wheels are on the kerb.

5. Now lift the handgrips and move forward to place the rear wheels on the kerb.
6. When the rollator is standing stable, you can take the kerb yourself.
7. To step down the kerb, act in the same manner. If the kerb is too high, take it backwards. Always make sure that at least two wheels are on the ground.

## 6 **Maintenance**

**i** Regular care ensures that your rollator is preserved in a perfectly functional condition.

### 6.1 *Cleaning*

Use a soft, damp cloth and lukewarm water with a mild soap for cleaning of the rollator. Avoid soaking.  
 Do not use abrasive cleaning agents to clean.

### 6.2 *Disinfection*

 **CAUTION** **Risk of damage to the rollator**  
**Disinfections may only be undertaken by trained persons. Consult your specialist dealer.**

### 6.3 *Inspection / Service / Repairs*

The lifespan of the rollator is influenced by its use, storage, maintenance, service and cleaning.

Before you resume use of the rollator, it should be inspected on following points:

- Completeness.
- Folding mechanism.
- Bolt and nuts regularly; on tightness.
- Check castors (stability, tracking, fixing screws).  
 In case of regular contact with wires, hair or other residues, the wheels may get blocked. Check regularly that the wheels rotate freely and that they are in good condition.
- Frame (deformation, stability, connections).
- Handgrip (safe locking of height adjustment system, good support of handgrips).

- Seat, backrest and storage bag (wear or damage).
- Being clean.
- Brakes: check regularly the working of the brakes.

The service frequency depends on the frequency and intensity of use. Consult your specialist dealer to make an agreement on service frequency.

Repairs and replacements may only be undertaken by trained persons and only genuine replacement parts of Vermeiren should be used. Do not hesitate to use the services offered by your specialist dealer. He/she will be glad to assist you regarding any service and repairs.

**i** The last page of this manual contains the service registration list to be used by specialist dealer. Additional technical information for the specialist dealer is available via our website:  
<http://www.vermeiren.com/>

## 7 Disposal

Packaging materials can be disposed at the recycling centre in your neighbourhood.

At end of life, you need to dispose your rollator according to the local environmental legislation. The best way to do so, is to disassemble the rollator to facilitate the transport of recyclable parts.

## 8 Technical details

 CAUTION Risk of unsafe use

Be aware of the correct settings and limits of use like described in this manual

<b>Make</b>	Vermeiren
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout Belgium
<b>Type</b>	Rollator 
<b>Model</b>	Eco-Plus

<i>Description</i>	<i>Dimensions</i>
Maximum user weight	136 kg
Maximum weight of basket load	5 kg
Dimensions unfolded (L) x W X (H)	(680-795) x 610 x (845-985) mm
Dimensions folded (handgrips in lowest position) L x W X H	310 x 610 x 960 mm
Distance between handgrips (centrelines)	500 mm
Seat height	555 - 625 mm
Seat width	360 mm
Seat depth	360 mm
Turning width	940 mm
Total mass	7,2 kg
Wheels (front & rear)	200 x 35 mm
Brakes	Running & parking brakes
Max. inclination forwards	15°
Max. inclination backwards	7°
Max. inclination sideways	3,5°
Storage temperature	+5°C - +41°C

We reserve the right to introduce technical changes.

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## Service registration form

This product (name): .....

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:
By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:	By (stamp):  Kind of work: I / S / R / D Date:

## Belgium

N.V. Vermeiren N.V.

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: [www.vermeiren.be](http://www.vermeiren.be)

e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

Vermeiren France S.A.

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59133 Phalempin

phone: +33(0)3 28 55 07 98

fax: +33(0)3 20 90 28 89

website: [www.vermeiren.fr](http://www.vermeiren.fr)

e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

Vermeiren Italia

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I-20020 Arese MI

phone: +39 02 99 77 07

fax: +39 02 93 58 56 17

website: [www.vermeiren.it](http://www.vermeiren.it)

e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## Poland

Vermeiren Polska Sp. z o.o.

ul. Łączna 1

PL-55-100 Trzebnica

phone: +48(0)71 387 42 00

fax: +48(0)71 387 05 74

website: [www.vermeiren.pl](http://www.vermeiren.pl)

e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132

702 00 Ostrava 1

phone: +420 596 133 923

fax: +420 596 121 976

website: [www.vermeiren.cz](http://www.vermeiren.cz)

e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

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D-40472 Düsseldorf

phone: +49(0)211 94 27 90

fax: +49(0)211 65 36 00

website: [www.vermeiren.de](http://www.vermeiren.de)

e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4

A-4061 Pasching

phone: +43(0)7229 64900

fax: +43(0)7229 64900-90

website: [www.vermeiren.at](http://www.vermeiren.at)

e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

Vermeiren Suisse S.A.

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3645 Gwatt (Thun)

phone: +41(0)33 335 14 75

fax: +41(0)33 335 14 67

website: [www.vermeiren.ch](http://www.vermeiren.ch)

e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5

Sant Gregori Parc Industrial Edifici

A 17150 Sant Gregori (Girona)

phone: +34 972 42 84 33

fax: +34 972 40 50 54

website: [www.vermeiren.es](http://www.vermeiren.es)

e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)

## Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: [www.vermeiren.com](http://www.vermeiren.com)

e-mail: [info@vermeiren.com](mailto:info@vermeiren.com)

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### **All rights reserved, including translation.**